# PRODUCT DATA SHEET STERILIZATION WRAPS

## STERISHEET DUO RANGE



## DESCRIPTION

Sterisheet sterilization wraps are best in class Sterile Barrier Systems for CSSDs in hospitals and clinics.

Sterisheet SMS products are available as interleaved, sealed or single sheets.

**STERISHET DUO** by Sterisheet is patented, unique combination of two thermally sealed products.

Designed as sealed combination of SMS and high quality cellulose based product to ensure the best bacterial barrier and wet pack prevention.

Double non-sequential wrapping with Sterisheet DUO is performed in one single step, saves time during

wrapping and unwrapping.

## **COMPOSITION**

The selected combination of SMS and cellulose based product:

- Crepe
- Reinforced
- Non Woven



Combo of two different colors (green & blue) for effective and easy defects detection.

3 grades for different strength level (100 gsm, 115 gsm, 120 gsm).

## SUITABLE FOR STERILIZATION METHODS

- o Steam
- **EO**

#### **APPLICATION**

For small, medium-size and heavy trays depending on the grade selected.



## PRODUCT DATA SHEET STERISHEET STERILIZATION WRAPS

## STERISHEET DUO RANGE



214

## SIZES AVAILABLE

Choose standard sizes to optimize your costs.

	STANDARD SIZE			
	90x90	100x100	120x120	137x137
PRODUCT NAME	cm	cm	cm	cm
Sterisheet Duo 1	$\checkmark$	$\checkmark$	$\checkmark$	
Sterisheet Duo 2	$\checkmark$	$\checkmark$	$\checkmark$	
Sterisheet Duo 3	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$

Other sizes are also available upon request.

## COLORS



#### PERFORMANCES

Mechanical and bacterial barrier properties are internally tested on a routine basis according to the regulatory requirements of all of our products. Combining them with random tests conducted by external and accredited laboratories lead Sterimed to secure the best performances on Sterisheet products.

#### **Mechanical Properties**

Preservation of pack integrity from closure until the point of use depends on the materials resistance to tearing, puncturing and breaching stresses generated all along the distribution with the hospital. Any mechanical weaknesses will increase the risks of event-related ingression of microorganisms into the pack. Excellent mechanical properties will provide you additional safety while using our materials. Optimal strength and resistance provided in every sheet.



sterimed.f

STERISHEET ® IS A BRAND OF

W

W



## PRODUCT DATA SHEET STERILIZATION WRAPS

## STERISHEET DUO RANGE



Higher the number used in products' name (from Duo 1 to Duo 3) higher the total grammage of the product and stronger the wrap.

## **Bacterial Properties**

Sterilization wraps must prevent microorganisms' ingression inside the package. To reach this performance, different types of testing have to be performed on the products to reproduce both kind of ingression vehicles possible:

• Airborne ingression

• Waterborne ingression

## **COMPLIANCE TO STANDARDS**

Sterisheet products range is classified as a Class I Medical Device according to the European Medical Device Device Regulation (MDR 2017/745/EU). Its CE marking illustrates the relevant compliance. Sterisheet SMS products conform to the standards below: EN ISO 11607-1 EN 868-2

## **OUR WRAPS MANUFACTURING CERTIFICATION**

ISO 13485 standard

### PACKAGING PRIOR TO USE

Sheets displays is optimized by adjusted folding depending on the size and type of the product. We have carefully tested the best solution to ensure the most convenient handling for end users.

PRIMARY TRANSPORT PACKAGING

W

W

Number of sheets is optimized and wrapped in transparent polyethylene bag with quick product ID.

SECONDARY TRANSPORT PACKAGING

Secondary packaging is a neutral brown color cardboard box providing transportation stress resistance.

Product Data Sheet - STERISHEET DUO RANGE | VERSION 2 | Edited on July 14 - 2021

sterimed.fr

STERISHEET ® IS A BRAND OF



# PRODUCT DATA SHEET STERILIZATION WRAPS

## STERISHEET DUO RANGE



## LABELING

Product traceability is fully insured through labelling according regulations on each transport packaging.

## **STORAGE CONDITIONS**

Sterimed recommends the following storage conditions for best performance of sterilization wraps: Storage in

cool, dry location away from direct exposure to natural light, strong artificial light & UV sources. Cardboard boxes should never be stored in direct contact with the floor. Storage of the products shall be done in areas that are not subject to extreme temperature changes such as in contact with heated objects, vents or cold walls.

As per AAMI ST79 "*Comprehensive guide to steam sterilization and sterility assurance in health care facilities*" recommendations: Before use, hold packaging materials at room temperature (20°C to 23°C) and a relative humidity ranging from 30% to 60% for a minimum of 2 hours is a good practice for optimum use performances.

## **USE BY DATE**

Provided the above storage conditions are met, the upper limit of the time interval during which the performance characteristics of sterilization wrap are demonstrated is 5 years of the manufacturing date.

#### **ENVIRONMENTAL IMPACT & WASTE MANAGEMENT**

Combination of cellulose based product with renewable content rate up to 97-100% and oil derived product, renewable content 0%.

Disposal as per local regulations after use.

